


# Fracture Management

## Product Guide

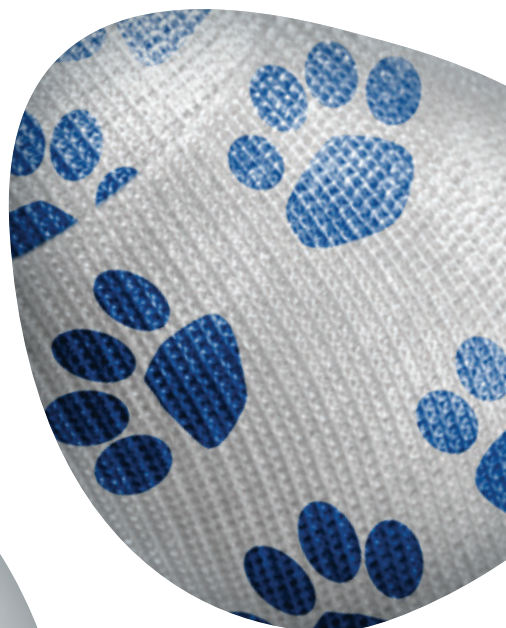

**Delta-Cast®**





## About Essity

**Essity** is a leading global health and medical solutions provider specialising in the areas of Wound Care, Fracture Management, Physiotherapy, Compression and Continence Care. We are focused on the development of world-class, branded products that offer high quality solutions for clinicians and patients alike.

For more product information and advice please contact our Customer Service team who are always ready to answer your queries or take order details by telephone **0845 1223600**, or by email **[orders.uk@essity.com](mailto:orders.uk@essity.com)**



# Contents

## Casting Tapes and Splinting

### Non-Fibreglass Casting Tape

Delta-Cast® Conformable	5
Delta-Cast® Elite	6
Delta-Cast® Prints	7

### Fibreglass Casting Tape

Delta-Lite® Plus	8
------------------	---

### Flexible Casting Tape

Delta-Cast® Soft	9
------------------	---

### Acute Synthetic Splint Systems

Dynacast® Prelude	10
Dynacast® Prelude Solo	10
Dynacast® Prelude Strap Kits	11
Dynacast® AS Ankle Splint	11
Delta-Xpress®	12

### Plaster of Paris

Gypsona®	14
Specialist® E	14
Biplatrix®	15

## Undercast Padding

### Padding

Soffban® Natural	17
Velband®	17
Soffban® Synthetic	18
Delta-Rol® Synthetic	18
Delta-Rol® 'S' Synthetic	19

### Water Resistant Cast Padding

Delta-Dry®	19
Delta-Dry® Pantaloons	20

## Stockinette

### Natural

Jersey Standard	22
Delta-Net® Stockinette	22

### Synthetic

Delta Terry-Net® C	23
Delta Terry-Net® S	23
Delta Terry-Net® Thumb Spica	23

## Thermoplastics

### Thermoplastics

Delta Thermo®	25
---------------	----

## Casting Accessories

### Accessories

Tensoban® Pre-Tape Underwrap	27
Tensoplast® Sport Cast Edge Tape	27
Delta Terry-Net® Adhesive Felt Liner	28
Delta Terry-Net® Cloth and Foam Liner	28
Delta Terry-Net® Adhesive Fleece Liner	29
Delta Terry-Net® Adhesive Hook and Stretch Loop Fabric	29

### Cast Shoes

Solo® Vinyl Cast Shoe	30
Eco Canvas Cast Sandal	30

### Skin Traction Kits

Tensoplast® Skin Traction Kits	31
--------------------------------	----





## Casting Tapes and Splinting

As part of the globally successful Essity health and medical solutions portfolio, the Delta-Cast® brand of casting tapes has the resources, reach and commitment to ensure you receive comprehensive support.

We are committed to building close, long-standing working relationships with our customers. Through these relationships we are able to learn valuable insights into the views and concerns of leading professionals, helping guide our efforts to improve our products and services. This cooperation allows us to develop relevant and innovative, high quality products and services to support healthcare professionals to take care of their patients.

Our commitment to professional education is illustrated with our Essity Academies programme. It underpins our belief that progress in patient care depends on continuing professional development and our educational initiatives are an integral part of our relationship with clinicians.

### **The Academies were launched to offer...**

- Therapy focused training modules
- Flexibility to run local bespoke clinical study sessions
- Modules in fracture management, wound care, compression and continence care

To find out more please email [essityacademies.uk@essity.com](mailto:essityacademies.uk@essity.com) or call the hotline on **0845 1200033**



# Delta-Cast® Conformable



## Non-Fibreglass Cast Tape

**Delta-Cast® Conformable** has set a benchmark in standard and removable, non-fibreglass cast tapes. Delta-Cast® Conformable responds to the daily challenges of fracture management - it's distinctive construction provides rigid and semi-rigid casting options for primary and secondary casting applications where patient fit, comfort and compliance are critical.

Delta-Cast® Conformable is the only UK cast tape with clinical evidence<sup>1</sup> to support its use for focused rigidity casting.

Code	Size	Colour	Pack Qty	NHS SC Code
72280-00	2.5cm x 1.80m	○ White	1 x 10	EAF2659
72280-01	5cm x 3.6m	○ White	1 x 10	EAF308
72280-02	7.5cm x 3.6m	○ White	1 x 10	EAF401
72280-03	10cm x 3.6m	○ White	1 x 10	EAF402
72280-04	12.5cm x 3.6m	○ White	1 x 10	EAF403
72281-00	5cm x 3.6m	● Pink	1 x 10	EAF495
72281-01	7.5cm x 3.6m	● Pink	1 x 10	EAF510
72281-02	10cm x 3.6m	● Pink	1 x 10	EAF521
72282-00	5cm x 3.6m	● Dark blue	1 x 10	EAF492
72282-01	7.5cm x 3.6m	● Dark blue	1 x 10	EAF504
72282-02	10cm x 3.6m	● Dark blue	1 x 10	EAF516
72283-00	5cm x 3.6m	● Yellow	1 x 10	EAF498
72283-01	7.5cm x 3.6m	● Yellow	1 x 10	EAF513
72283-02	10cm x 3.6m	● Yellow	1 x 10	EAF525
72284-00	5cm x 3.6m	● Orange	1 x 10	EAF494
72284-01	7.5cm x 3.6m	● Orange	1 x 10	EAF508
72284-02	10cm x 3.6m	● Orange	1 x 10	EAF518
72285-00	5cm x 3.6m	● Red	1 x 10	EAF497
72285-01	7.5cm x 3.6m	● Red	1 x 10	EAF512
72285-02	10cm x 3.6m	● Red	1 x 10	EAF524
72286-00	5cm x 3.6m	● Purple	1 x 10	EAF496
72286-01	7.5cm x 3.6m	● Purple	1 x 10	EAF511
72286-02	10cm x 3.6m	● Purple	1 x 10	EAF523
72287-00	5cm x 3.6m	● Green	1 x 10	EAF493
72287-01	7.5cm x 3.6m	● Green	1 x 10	EAF506
72287-02	10cm x 3.6m	● Green	1 x 10	EAF517
72288-00	5cm x 3.6m	● Black	1 x 10	EAF491
72288-01	7.5cm x 3.6m	● Black	1 x 10	EAF502
72288-02	10cm x 3.6m	● Black	1 x 10	EAF515
72289-10	5cm x 3.6m	Mixed†	1 x 10	EAF490
72289-11	7.5cm x 3.6m	Mixed†	1 x 10	EAF665
72289-12	10cm x 3.6m	Mixed†	1 x 10	EAF514

†Mixed packs contain two rolls each of dark blue, black, red, purple and pink

### High performance 3-D stretch polyester bandage

- Outstanding conformability allows uninterrupted application without the need to tuck and fold, and without creasing
- 3-D stretch helps to cover even difficult body contours in one go to help produce wrinkle-free casts, without pressure points
- Long-lasting durability and resiliency for minimal cast breakdown

### Patented resin formula

- Silicone and latex-free resin is designed to support ease of application and to ensure strong, long lasting lamination, offering:
  - Consistent and reliable set times
  - High lamination of layers and end lay down for optimum stability, strength and reliability
  - A smooth sensation during moulding which results in a smooth finish to the cast
  - Application with any standard examination gloves

### Countless options

- Versatile rigidity enables the practitioner to generate either traditional total contact casts (TCC) or focused rigidity casts (FRC) with just one product
- Removable casts can be made and re-applied for greater control of the healing process



1. Petty, A. C., Wardman, C., (1998). A randomized, controlled comparison of adjustable focused rigidity primary casting technique with standard plaster of Paris/synthetic casting technique in the management of fractures and other injuries. Journal of Orthopaedic Nursing. 2. 95 - 102



## Non-Fibreglass Cast Tape

Used for primary and secondary casting applications, **Delta-Cast® Elite** has been designed with ease of application and patient comfort in mind. Our patented non-fibreglass casting material combines strength and rigidity with soft edges for patient comfort.

### Non-fibreglass

- Provides smooth and soft cast edges for enhanced patient comfort

### Highly conformable polyester casting material

- High strength, rigidity and durability
- Easily conforms to body contours without folding or tucking for increased patient comfort
- Smooth finish of the cast reduces potential snag on clothing and irritation of the skin

### Consistent set-time

- Sets in approximately three minutes and is fully weight bearing after 20 minutes

Code	Size	Colour	Pack Qty	NHS SC Code
72279-00	5cm x 3.6m roll	○ White	1 x 10	EAF385
72279-01	7.5cm x 3.6m roll	○ White	1 x 10	EAF386
72279-02	10cm x 3.6m roll	○ White	1 x 10	EAF387
72279-03	12.5cm x 3.6m roll	○ White	1 x 10	EAF388
72278-00	5cm x 3.6m roll	● Black	1 x 10	EAF398
72278-01	7.5cm x 3.6m roll	● Black	1 x 10	EAF399
72278-02	10cm x 3.6m roll	● Black	1 x 10	EAF400
72276-00	5cm x 3.6m roll	● Dark blue	1 x 10	EAF389
72276-01	7.5cm x 3.6m roll	● Dark blue	1 x 10	EAF390
72276-02	10cm x 3.6m roll	● Dark blue	1 x 10	EAF391
72277-00	5cm x 3.6m roll	● Purple	1 x 10	EAF395
72277-01	7.5cm x 3.6m roll	● Purple	1 x 10	EAF396
72277-02	10cm x 3.6m roll	● Purple	1 x 10	EAF397
72275-00	5cm x 3.6m roll	● Red	1 x 10	EAF392
72275-01	7.5cm x 3.6m roll	● Red	1 x 10	EAF393
72275-02	10cm x 3.6m roll	● Red	1 x 10	EAF394
73455-00	5cm x 3.6m roll	● Pink	1 x 10	EAF534
73455-01	7.5cm x 3.6m roll	● Pink	1 x 10	EAF535
73455-02	10cm x 3.6m roll	● Pink	1 x 10	EAF536
73453-00	5cm x 3.6m roll	● Light blue	1 x 10	EAF584
73453-01	7.5cm x 3.6m roll	● Light blue	1 x 10	EAF585
73453-02	10cm x 3.6m roll	● Light blue	1 x 10	EAF586
73454-00	5cm x 3.6m roll	● Green	1 x 10	EAF587
73454-01	7.5cm x 3.6m roll	● Green	1 x 10	EAF588
73454-02	10cm x 3.6m roll	● Green	1 x 10	EAF589
73456-00	5cm x 3.6m roll	● Orange	1 x 10	EAF590
73456-01	7.5cm x 3.6m roll	● Orange	1 x 10	EAF591
73456-02	10cm x 3.6m roll	● Orange	1 x 10	EAF592


















# Delta-Cast® Prints

## Non-Fibreglass Cast Tape

**Delta-Cast® Prints** is a range of colourful alternative patterned non-fibreglass cast tapes which give patients a variety of design options for their casts. The colourful designs can also help to overcome the anxiety of young and old patients in cast rooms.

Code	Size	Colour	Pack Qty
72273-22	5cm x 3.6m	 Army camouflage	1 x 10
72273-06	7.5cm x 3.6m	 Army camouflage	1 x 10
72273-02	5cm x 3.6m	 Dinosaur print	1 x 10
72273-07	7.5cm x 3.6m	 Pastel print	1 x 10
72273-08	5cm x 3.6m	 Football print	1 x 10
72273-09	7.5cm x 3.6m	 Football print	1 x 10
72273-05	5cm x 3.6m	 Teddy bear print	1 x 10
72273-20	5cm x 3.6m	 Paw print	1 x 10
72273-21	7.5cm x 3.6m	 Paw print	1 x 10
72273-03	5cm x 3.6m	 Winter print	1 x 10
72273-04	7.5cm x 3.6m	 Winter print	1 x 10
72273-40	5cm x 3.6m	 Pink camouflage	1 x 10
72273-41	7.5cm x 3.6m	 Pink camouflage	1 x 10

### Colourful printed pattern

- The colourfast playful patterns make wearing a cast more fun and can help increase patient compliance

### Highly conformable polyester casting material

- The high conformability supports confident, wrinkle-free, precise application, with even unrolling tension and good end lay down with secure lamination for long-term durability

### Reliable and consistent working time


- The reliable and consistent working time window delivers a fully weight bearing cast after approximately 20 minutes

### Robust, lightweight, smooth cast

- Rigid yet flexible material helps prevent constriction of muscle and tissue
- The smooth finish of the cast reduces snag on clothing and irritation of the skin

### Available in eight patterns

- Providing the patient with a wide variety of choice

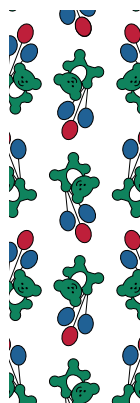



## Fibreglass Cast Tape

**Delta-Lite® Plus** is a smooth, durable, fibreglass cast tape that combines a fibreglass substrate and a polyurethane resin system with tack free properties. The tack-free properties allow ease of application and moulding using standard examination gloves.

### Smooth finish surface

- Non-abrasive finished cast enhances patient comfort

### Excellent rigid support

- Creates a long-lasting, lightweight cast

### High lamination

- Gives strength between cast layers
- Improved cast strength

### Good conformability

- Makes casting application easier around all body contours
- No special application techniques required

### Outstanding moulding

- Provides excellent anatomical support

Code	Size	Colour	Pack Qty	NHS SC Code
73458-00	2.5cm x 1.8m roll	○ White	1 x 10	EAF435
73458-01	5cm x 3.6m roll	○ White	1 x 10	EAF443
73458-02	7.5cm x 3.6m roll	○ White	1 x 10	EAF450
73458-03	10cm x 3.6m roll	○ White	1 x 10	EAF463
73458-04	12.5cm x 3.6m roll	○ White	1 x 10	EAF466
73458-55	5cm x 3.6m roll	● Pink	1 x 10	EAF440
73458-56	7.5cm x 3.6m roll	● Pink	1 x 10	EAF447
73458-57	10cm x 3.6m roll	● Pink	1 x 10	EAF457
73458-20	5cm x 3.6m roll	● Dark blue	1 x 10	EAF437
73458-21	7.5cm x 3.6m roll	● Dark blue	1 x 10	EAF451
73458-22	10cm x 3.6m roll	● Dark blue	1 x 10	EAF453
73458-30	5cm x 3.6m roll	● Red	1 x 10	EAF442
73458-31	7.5cm x 3.6m roll	● Red	1 x 10	EAF449
73458-32	10cm x 3.6m roll	● Red	1 x 10	EAF461
73458-60	5cm x 3.6m roll	● Purple	1 x 10	EAF441
73458-61	7.5cm x 3.6m roll	● Purple	1 x 10	EAF448
73458-62	10cm x 3.6m roll	● Purple	1 x 10	EAF459
73458-35	5cm x 3.6m roll	● Light blue	1 x 10	EAF438
73458-36	7.5cm x 3.6m roll	● Light blue	1 x 10	EAF445
73458-37	10cm x 3.6m roll	● Light blue	1 x 10	EAF454
73458-45	5cm x 3.6m roll	● Black	1 x 10	EAF436
73458-46	7.5cm x 3.6m roll	● Black	1 x 10	EAF444
73458-47	10cm x 3.6m roll	● Black	1 x 10	EAF452
73458-25	5cm x 3.6m roll	● Green	1 x 10	EAF439
73458-26	7.5cm x 3.6m roll	● Green	1 x 10	EAF446
73458-27	10cm x 3.6m roll	● Green	1 x 10	EAF455
73458-65	5cm x 3.6m roll	● Yellow	1 x 10	EAF531
73458-66	7.5cm x 3.6m roll	● Yellow	1 x 10	EAF532
73458-67	10cm x 3.6m roll	● Yellow	1 x 10	EAF533
73458-75	5cm x 3.6m roll	● Orange	1 x 10	EAF2602
73458-76	7.5cm x 3.6m roll	● Orange	1 x 10	EAF2603
73458-77	10cm x 3.6m roll	● Orange	1 x 10	EAF2604





## Flexible Casting Tape

**Delta-Cast® Soft** is designed to provide functional support to fractures and soft tissue injuries. It's semi-rigidity allows for flexibility and mobility, helping to reduce immobilisation related problems such as muscle atrophy and joint stiffness. Improved flexibility and mobility can help promote a faster healing process.

Delta-Cast® Soft is constructed using a combination of high-tech polyester, polypropylene, and elastane yarns, together with a specially developed resin. This technology builds safety, functionality and comfort into a product offering all the advantages only a non-fibreglass casting tape can offer.

Code	Size	Colour	Pack Qty	NHS SC Code
72708-02	2.5cm x 1.8m roll	○ White	1 x 10	EAF659
72708-12	5cm x 3.6m roll	○ White	1 x 10	EAF660
72708-22	7.5cm x 3.6m roll	○ White	1 x 10	EAF661
72708-32	10cm x 3.6m roll	○ White	1 x 10	EAF657
72708-42	12.5cm x 3.6m roll	○ White	1 x 10	EAF658
72708-05	5cm x 3.6m roll	● Blue	1 x 10	EAF650
72708-15	7.5cm x 3.6m roll	● Blue	1 x 10	EAF651
72708-25	10cm x 3.6m roll	● Blue	1 x 10	EAF649
72708-08	5cm x 3.6m roll	● Red	1 x 10	EAF655
72708-18	7.5cm x 3.6m roll	● Red	1 x 10	EAF656
72708-28	10cm x 3.6m roll	● Red	1 x 10	EAF654
72708-50	5cm x 3.6m roll	● Pink	1 x 10	EAF663
72708-51	7.5cm x 3.6m roll	● Pink	1 x 10	EAF664
72708-52	10cm x 3.6m roll	● Pink	1 x 10	EAF662
72708-01	5cm x 3.6m roll	● Black	1 x 10	EAF647
72708-11	7.5cm x 3.6m roll	● Black	1 x 10	EAF648
72708-21	10cm x 3.6m roll	● Black	1 x 10	EAF646
72708-09	5cm x 3.6m roll	● Paw print	1 x 10	EAF652
72708-19	7.5cm x 3.6m roll	● Paw print	1 x 10	EAF653
72708-60	5cm x 3.6m roll	● Heart print	1 x 10	EAF668
72708-61	7.5cm x 3.6m roll	● Heart print	1 x 10	EAF669



### Three-way stretch / excellent conformability

- Ease of handling on application
- Allows for a consistent fit around difficult body contours with a wrinkle-free application, offering enhanced fit and support for the patient

### Non-fibreglass

- Highly durable to maintain cast functionality, helping reduce time and costs for the cast room that can be associated with cast repairs

### Range of colours and unique designs

- Available in patented paw print, heart or colour designs, offering patients of all ages a more playful, pleasant cast
- Helps support patient concordance and function

### Versatile application

- Designed for both customised removable casts and circular applications, Delta-Cast® Soft can easily be trimmed to shape, producing an optimal fit for the patient
- Can be combined with hook and loop straps for removable applications, or with reinforcement materials for stronger support

### 100% Radiolucent

- Allows clear X-ray images for diagnosis, offering cost and time savings

### Flexibility

- Can help to minimise some negative effects of total rigid immobilisation; muscle atrophy, joint stiffness and other immobilisation related issues
- Improved circulation can help towards a faster healing process and a shorter rehabilitation period

### Ease of removal

- Can be removed by lifting and unpeeling
- No cast saw required. Can be removed using regular cast scissors

### Excellent lamination

- Excellent lamination of layers and good end lay down produces a strong, well bonded cast



# Dynacast® Prelude

LATEX  
FREE  
FORMULA

## Acute Synthetic Splint System

**Dynacast® Prelude** is an acute fibreglass splinting system specifically incorporating fibreglass, covered by a polypropylene padding. Stronger and more versatile than traditional plaster of Paris, it is ideal for trauma and post-op applications from acute through to rehabilitation phase.

### Strong, light and durable

- Resin coated fibreglass provides a high strength splint which is resistant to breakdown
- Will not breakdown when wet

### Quick setting and early weight bearing

- Dynacast® Prelude has an initial set of 3 - 5 minutes and reaches weight bearing strength in approximately 20 minutes

### No mess or waste

- With no plaster mess, Dynacast® Prelude is ideally suited to the theatre and ward environment
- The continuous roll format ensures clinicians will only use what is required and provides versatility to choose the length of splint to suit the indication and patient

### Patient comfort

- Soft polypropylene padding covers the fibreglass splint minimising the risk of pressure points
- Naturally aids moisture transmission away from the skin to enhance patient comfort

Code	Size	Colour	Pack Qty	NHS SC Code
71418-00	2.5cm x 4.6m roll	White	1 x 1	EAFO82
71420-00	5cm x 4.6m roll	White	1 x 1	EAFO83
71421-00	7.5cm x 4.6m roll	White	1 x 1	EAFO84
71415-00	10cm x 4.6m roll	White	1 x 1	EAFO85
71416-00	12.5cm x 4.6m roll	White	1 x 1	EAFO86
71417-00	15cm x 4.6m roll	White	1 x 1	EAFO87
71419-00	20cm x 2.3m roll	White	1 x 1	EAM1844

### ILS (interlocking strength) technology

- Provides strength and durability
- Minimises wrinkles and creases in the finished splint

# Dynacast® Prelude Solo

LATEX  
FREE  
FORMULA

## Unpadded Precut Splinting System

**The Dynacast® Prelude Solo** unpadded precut is a fibreglass splinting system that comes in a variety of widths and lengths, and is stronger and more versatile than traditional plaster of Paris slabs. Dynacast® Prelude Solo splints are easy to apply and clean, and are ideal for treating trauma and post-operative applications.

### Fibreglass splint

- Stronger than traditional plaster of Paris slabs
- Reduces weight and bulk to patient

### Simple activation with no plaster dust

- Liner enables gloveless application
- No dust and mess associated with traditional plaster of Paris slabs
- Ideal for use in theatre and A&E environments

### Slim profile

- Excellent mouldability

### ILS (interlocking strength) technology

- Provides strength and durability
- Minimises wrinkles and creases in the finished splint

Code	Size	Colour	Pack Qty	NHS SC Code
72963-00	5cm x 25cm	White	1 x 10	EAFO82
72963-01	7.5cm x 30cm	White	1 x 10	EAFO83
72963-02	7.5cm 89cm	White	1 x 10	EAFO84
72963-03	10cm x 38cm	White	1 x 10	GRS000
72963-04	10cm x 76cm	White	1 x 10	EAM1843
72963-05	12.5cm x 76cm	White	1 x 10	EAFO87
72963-06	15cm x 30cm	White	1 x 10	EAFO88





# Dynacast® Prelude Strap Kits

## Acute Synthetic Splint System

Designed for use with **Dynacast® Prelude** splints in the rehabilitation phase, this patient-friendly product enables long-term use without the need for a full bandage, enabling easy splint removal. Each kit contains three latex-free hook and loop straps.

Code	Size	Colour	Pack Qty
71415-18	Upper extremity kit	Red	1 x 10
71415-19	Lower extremity kit	Blue	1 x 10



# Dynacast® AS Ankle Splint

LATEX  
FREE  
FORMULA

## Acute Synthetic Splint System

**Dynacast® AS Ankle Splint** is a custom mouldable solution for external ankle support following acute injury or for chronic joint instability. This is a therapy solution that can be used throughout treatment from acute phase to rehabilitation.

Code	Size	Colour	Pack Qty
71984-00	Paediatric kit	blue	1 x 1
71361-00	Small / medium kit	blue	1 x 1
71360-00	Medium / large kit	Blue	1 x 1
72834-00	Small / medium dispenser pack	Blue	1 x 10
72835-00	Medium / large dispenser pack	Blue	1 x 10

### Dispenser pack of 10 splints

- 10 splints and straps
- Improved storage



### Low profile

- Allows the use of normal footwear
- Fits easily into training shoes

### Custom mouldable splint

- Allows for a precise fit for good patient comfort

### Pre-made fibreglass stirrup splint with micro-perforated foam covering

- Ideal for A&E
- Strong, lightweight, breathable yet durable therapy solution

### All in one treatment kit from acute to rehabilitation

- Oedema control
- Circumferential support
- Cold therapy in the acute stage
- U-shaped foam inserts



# Delta-Xpress®

## Splint System

**Delta-Xpress®** is a groundbreaking splinting system that allows limb immobilisation in a fraction of the time, compared to conventional splints or casts.

Delta-Xpress® offers four different designs to immobilise lower leg, arm, wrist and finger fractures. The prefabricated design is intuitive to use, while providing secure and precise immobilisation for short-term use.

### Precise clinical positioning

- Pre-defined angles hold limb in optimal healing position
- Available in three sizes

### Ready to use from the shelf

- Prefabricated design allows instant application
- No activation, no curing, no clean-up
- No need for water activation or even working space
- Instructions and icons support quick and correct application

### Secure – safe and strong

- Rigid compound material
- Good conformability for individual fit
- Fixation with bandage or hook and loop straps

### High performance design features

- Cushioned - padding inside for comfort and absorption
- Breathability - to help avoid skin maceration
- Size and shape - adjustable with standard scissors
- Convenient - splints come in three sizes with measuring scale and colour code on product
- Easy access to wounds

# Delta-Xpress® Wrist and Finger

## Wrist and Finger Splint

- Rigid stay for optimal wrist, hand and finger position
- The stay has an angle of 100° to support the flexion of the metacarpophalangeal joint (MCP) and stabilises the wrist in 30° dorsal extension

Code	Size	Pack Qty	NHS SC Code
72713-00	Small	1 x 10	EAZ153
72713-01	Medium	1 x 10	EAZ154
72713-02	Large	1 x 10	EAZ155





# Delta-Xpress® Wrist

## Wrist Splint

Code	Size	Pack Qty	NHS SC Code
72712-00	Small	1 x 10	EAZ150
72712-01	Medium	1 x 10	EAZ151
72712-02	Large	1 x 10	EAZ152

- Rigid stay for optimal wrist position
- Wrist angle in 30° dorsal extension guarantees a functional therapy position



# Delta-Xpress® Lower Leg

## Lower Leg Splint

Code	Size	Pack Qty	NHS SC Code
72711-00	Small	1 x 10	EAZ147
72711-01	Medium	1 x 10	EAZ148
72711-02	Large	1 x 10	EAZ149

- Pre-adjusted 90° angle for ideal healing position of lower leg
- Ankle and foot rest inlay for optimal foot position
- Malleoli paddings for increased comfort



# Delta-Xpress® Elbow

## Elbow Splint

Code	Size	Pack Qty	NHS SC Code
72710-00	Small	1 x 10	EAZ144
72710-01	Medium	1 x 10	EAZ145
72710-02	Large	1 x 10	EAZ146

- Pre-adjusted 90° angle assures optimal healing position of upper arm, elbow and lower arm





## Plaster of Paris

**Gypsona®** premium plaster of Paris bandage is made from a specially woven leno weave cloth, uniformly impregnated with high quality, creamy, fast setting plaster of Paris that produces a strong cast with an excellent finish. Available in roll and 5-ply slab format.

### Interlocked leno weave gauze

- Provides excellent stability and conformability to the wet bandage
- Minimises plaster loss
- Provides strong finished cast

### Smooth, creamy plaster formula

- Allows for a smooth aesthetically pleasing cast
- Improved moulding properties and excellent finish

### Nicked bandage edge design

- Ensures minimal fraying and loose threads during application
- Supports a smoother cast finish

### Round core

- Improved handling stability

Code	Size	Pack Qty	NHS SC Code
71985-04	5cm x 2.7m roll	1 x 48	EAA051
71985-05	7.5cm x 2.7m roll	1 x 48	EAA052
71985-00	10cm x 2.7m roll	1 x 48	EAA053
71985-02	15cm x 2.7m roll	1 x 24	EAA054
71985-03	20cm x 2.7m roll	1 x 24	EAA055
71988-14	10cm x 20m 5-ply slab dispenser	1 x 1	EAA023
71988-15	15cm x 20m 5-ply slab dispenser	1 x 1	EAA025
71988-16	20cm x 20m 5-ply slab dispenser	1 x 1	EAA027



# Specialist® E

## Plaster of Paris

**Specialist® E** plaster of Paris has a creamy texture resulting in a smooth feel, and making it easy to apply. It forms a well fitting cast that is reassuringly tough and strong. It is the ideal choice for casting applications where a quick setting time is required. Available in roll and 5-ply slab format.

### Smooth, creamy texture

- Results in a smooth, evenly finished cast
- Minimal plaster loss

### Highly mouldable and strong

- Produces an excellent fit around difficult body contours and delivers a well laminated cast

### Easy to apply and remove

- Able to 'window' the cast after only 10 minutes

### Moisture proof packaging

- Protects and prolongs the product shelf life

Code	Size	Pack Qty	NHS SC Code
72260-00	5cm x 2.7m roll	1 x 12	EAA056
72260-01	7.5cm x 2.7m roll	1 x 12	EAA057
72260-02	10cm x 2.7m roll	1 x 12	EAA058
72260-04	15cm x 2.7m roll	1 x 12	EAA049
72260-05	20cm x 2.7m roll	1 x 12	EAA050
72259-00	10cm x 20m 5-ply slab dispenser	1 x 1	EAA007
72259-02	15cm x 20m 5-ply slab dispenser	1 x 1	EAA008
72259-03	20cm x 20m 5-ply slab dispenser	1 x 1	EAA009





## Plaster of Paris

**Biplatrix®** is a very fast setting plaster of Paris bandage that consists of a cotton gauze substrate which is spread with gypsum and spooled on a round core. The bandages wet rapidly and evenly with no dry spots. The initial weight of plaster spread and it's low plaster loss combine to offer a smooth plaster and a durable cast.

Code	Size	Pack Qty	NHS SC Code
71995-24	5cm x 3m roll	1 x 40	EAA015
71995-05	8cm x 3m roll	1 x 40	EAA018
71995-06	10cm x 3m roll	1 x 40	EAA029
71995-09	15cm x 3m roll	1 x 40	EAA030
71995-11	20cm x 3m roll	1 x 20	EAA031
71995-15	10cm x 25m 4-ply slab dispenser	1 x 1	EAA032
71995-16	15cm x 25m 4-ply slab dispenser	1 x 1	EAA033
71995-17	20cm x 25m 4-ply slab dispenser	1 x 1	EAA034

### Ultra fast setting time

- Sets within two minutes when activated in water (at maximum 25°C)
- Ideal for orthotic applications

### High resistance and durability

- The high mass weight of Biplatrix® combined with low plaster loss provides a durable cast

### Strong plaster formula

- Less creamy than other plaster of Paris, Biplatrix® provides a strong rigid plaster

### Moisture resistant packaging

- Biplatrix® rolls are dual packed in a clear polythene film





## Undercast Padding

Essity offers a wide range of padding options for use in conjunction with both plaster of Paris and synthetic casting tapes. **Soffban® Natural** and **Velband®** offer the benefits of a natural padding with excellent absorption, making them both an ideal product for theatre use.

**Soffban® Synthetic**, **Delta-Rol®** and **Delta-Rol® S** are synthetic materials that are non-absorbent and therefore retain excellent cushioning when wet to give increased patient comfort. **Delta-Rol® S** also incorporates a super fine foam backing, which removes the need to use separate pre-tape for a smooth finish. **Delta-Dry®** water resistant cast padding has a unique patented technology, which allows patients to shower and bathe as normal when used with synthetic cast tapes.



# Soffban® Natural

LATEX  
FREE  
FORMULA

## Natural Viscose Padding

**Soffban® Natural** padding is suitable for use with both plaster of Paris and synthetic casts to aid comfort.

Code	Size	Pack Qty	NHS SC Code
71472-00	5cm x 2.7m roll	1 x 12	EPA071
71473-00	7.5cm x 2.7m roll	1 x 12	EPA073
71469-00	10cm x 2.7m roll	1 x 12	EPA075
71470-00	15cm x 2.7m roll	1 x 12	EPA077
71471-00	20cm x 2.7m roll	1 x 6	EPA079

### Highly absorbent natural viscose

- Suitable for use where bleeding may occur
- Wicks moisture away from the skin
- Reduces the risk of skin maceration and sensitivity

### Versatile

- Soffban® Natural is also suitable and widely used for venous leg ulcer multi-layer bandaging therapy as the first layer\*



### Easy to apply

- Excellent conformability around body contours
- Easy to tear
- Feathers easily and clings to itself
- Blends to give a smooth even finish

### Autoclavable

- Can be sterilised
- No need to remove the paper wrapping
- 100% viscose rayon

### Soft

- Comfortable for the patient, soft and gentle against the skin

### Excellent cushioning

- Padding structure creates a material which is light, deep and stable, with good airflow
- Protects from rigid casting material
- Also provides protection from the cast saw on cast removal

# Velband®

LATEX  
FREE  
FORMULA

## Natural Rayon Padding

**Velband®** padding is a natural rayon fibre padding, highly absorbent, drawing moisture away from the skin. It has a unique three-layer structure and is ideal for use in theatre where absorbent cushioning is required.

Code	Size	Pack Qty	NHS SC Code
72315-00	5cm x 2.75m roll	12 x 12	EPA011
72315-01	7.5cm x 2.75m roll	20 x 12	EPA013
72315-02	10cm x 2.75m roll	16 x 12	EPA015
72315-04	15cm x 2.75m roll	10 x 12	EPA017
72315-06	20cm x 2.75m roll	15 x 6	EPA019
72315-30	10cm x 4.5m roll	7 x 12	EPA014



### Unique construction

- Lightweight, breathable non-woven fabric
- The middle natural viscose layer gives excellent absorbent properties and cushioning

### Highly absorbent

- Suitable for use where bleeding may occur\*
- Absorbs exudate and moisture away from the skin
- Reduces the risk of skin maceration and sensitivity

### Excellent cushioning

- Unique three-layer construction provides excellent cushioning for improved patient comfort
- Reduces risk of pressure sores

### Autoclavable

- Can be sterilised for theatre use

\*A primary dressing should always be worn over open wounds



# Soffban® Synthetic

LATEX  
FREE  
FORMULA

## Synthetic Padding

**Soffban® Synthetic** padding is suitable for use with both plaster of Paris and synthetic casts.

### Synthetic non-absorbent material

- Wicks moisture away from the skin
- Reduces the risk of skin maceration and sensitivity
- Cushioning 'loft' is retained even if padding becomes wet

### Easy to apply

- Excellent conformability around body contours
- Easy to tear
- Feathers easily and clings to itself
- Blends to give a smooth even finish

### Excellent cushioning and soft feel

- Padding structure creates a material which is light, deep and stable, with good airflow
- Protects the patient with a reduced risk of skin abrasions
- Also provides protection from the cast saw on cast removal
- Comfortable for the patient, soft and gentle against the skin

Code	Size	Pack Qty	NHS SC Code
71464-01	5cm x 2.7m roll	1 x 12	EPA020
71467-01	7.5cm x 2.7m roll	1 x 12	EPA021
71486-05	10cm x 2.7m roll	1 x 12	EPA022
71465-01	15cm x 2.7m roll	1 x 12	EPA023
71466-01	20cm x 2.7m roll	1 x 6	EPA024

### Autoclavable

- Can be sterilised
- No need to remove the paper wrapping



# Delta-Rol®

LATEX  
FREE  
FORMULA

## Synthetic Padding

**Delta-Rol®** Synthetic padding is a non-absorbent synthetic material, it wicks moisture away from the skin and allows it to evaporate, resulting in a reduced risk of skin maceration. Suitable for use with plaster of Paris and synthetic casts.

### Synthetic non-absorbent material

- Wicks moisture away from the skin
- Reduces the risk of skin maceration and sensitivity
- Cushioning 'loft' of the padding is retained even when material becomes wet

### Easy to apply

- Good conformability around body contours
- Gives a smooth even finish under the cast

### Excellent cushioning and soft feel

- Padding structure creates a material which is light, deep and stable, with good airflow
- Protects the patient with a reduced risk of skin abrasions
- Also provides protection from the cast saw on cast removal
- Comfortable for the patient, soft and gentle against the skin

Code	Size	Pack Qty
72310-00	5cm x 2.75m roll	10 x 12
72310-01	7.5cm x 2.75m roll	24 x 12
72310-02	10cm x 2.75m roll	18 x 12
72310-03	15cm x 2.75m roll	12 x 12
72310-04	20cm x 2.75m roll	18 x 6

### Autoclavable

- Can be sterilised
- No need to remove the paper wrapping





## Synthetic Padding

**Delta-Rol® S Synthetic** is a unique combination of a synthetic undercast padding needled onto a thin polyurethane foam bandage. This exceptional combination delivers excellent conformability and cushioning together with the smooth, even finish associated with the use of a pre-tape over cast padding.

Code	Size	Pack Qty
72322-00	5cm x 2.75m roll	10 x 12
72322-01	7.5cm x 2.75m roll	12 x 12
72322-02	10cm x 2.75m roll	9 x 12
72322-03	15cm x 2.75m roll	6 x 12
72322-04	20cm x 2.75m roll	9 x 6



### Non-absorbent material

- Wicks moisture away from the skin
- Reduces the risk of skin maceration and sensitivity
- Cushioning 'loft' of the padding is retained even if material becomes wet

### Polyurethane foam lining

- Provides a smooth even finish on which to apply the cast

### Easy to apply

- Good conformability around body contours
- Gives a smooth even finish under the cast
- Separate pre-tape not required

### Excellent cushioning

- Padding structure creates a material which is light, deep and stable, with good airflow
- Protects the patient with a reduced risk of skin abrasions
- Also provides protection from the cast saw on cast removal

# Delta-Dry®

## Water Resistant Cast Padding

**Delta-Dry®** has a patented technology allowing patients to clean and rinse their cast and also bathe and shower as normal. Delta-Dry® is conformable and easy to apply, resulting in a comfortable cast.

Ideal for use with incontinent paediatric or elderly and less mobile patients when used with a synthetic cast tape.

Code	Size	Pack Qty	NHS SC Code
73443-00	5cm x 2.4m roll	1 x 12	EPA025
73443-01	7.5cm x 2.4m roll	1 x 12	EPA026
73443-02	10cm x 2.4m roll	1 x 12	EPA027



### Water-resistant cast padding

- Allows patients to shower and bathe as normal
- Increases hygiene
- Reduces odour
- Improves patient compliance when wearing a cast

### Easy to apply

- Good conformability around body contours
- No special tools are needed for application or removal

### How does it work?

- The open knit structure, along with the water-resistant qualities of the fibres, allows the water to drain from the cast
- The majority of the water simply runs out of the cast. The remaining moisture evaporates through the cast, heated by normal body temperature
- Position the cast to allow water to drain from it
- Long arm and leg casts must be positioned as to not allow water to pool in the vicinity of the heel or elbow
- Casts will take 30 minutes to 1 hour approximately to completely dry out



# Delta-Dry® Pantaloon

## The Hygienic Hip Spica Cast Treatment Solution

**Essity** continually strives to develop and offer the best solutions that support medical experts in achieving safe and successful treatments. In particular, the care of paediatric patients needs considered development as well as the products used in that treatment.

This pre-shaped, soft and water-resistant liner has been developed using the patented Delta-Dry® technology to simplify the treatment of developmental dysplasia of the hip or femur fractures. The aim of the Delta-Dry® Pantaloon is to help support improved patient hygiene and ease of application for both patient and clinician.

Delta-Dry® Pantaloon maintains patient hygiene, as flushing the hip spica cast and even bathing is possible. For best results, only use Delta-Dry® Pantaloon in combination with water-resistant, synthetic cast material.

### Delta-Dry technology

Water-resistant liner material that can help to increase patient and parent acceptance and potentially supports time and cost-savings

- Enables bathing and flushing
- Helps to avoid odour, itching and skin maceration
- Can help to allow expanded duration between cast changes (up to six weeks)

### Pre-cut shape in three sizes

- Permits an easy application, a suitable fit with little creasing offering high wearing comfort and allowing safe and easy application

### Quick dry system

High water vapour permeability for optimised hygienic care

- Dries within 90 minutes
- Helps patient's skin to breathe


### Care and protection

Soft, flexible and cushioned for infant skin to offer high wearing comfort

- Protects bony prominences
- Provides safety during cast removal
- Comes with foldable flap and cast ends

### X-ray translucency

Code	Size	Measurements	Pack Qty	NHS SC Code
73365-00	1	≤ 9.8" / ≤ 25 cm	1	EAF920
73365-02	2	9.8 - 11.6" / 25 - 29.5 cm	1	EAF921
73365-03	3	≥ 11.6" / ≥ 29.5 cm	1	EAF922

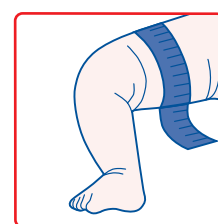

### Also available:

Delta-Dry® Pantaloon patient information brochure – a practical guide for parents whose child is receiving a Delta-Dry® Pantaloon cast treatment.





# Stockinette - Natural and Synthetic

Providing both natural and synthetic stockinette, Essity offers customers their product of choice. Stockinette can be used with both synthetic and plaster of Paris materials.

**Delta-Net®** and **Jersey Standard** are traditional stockinette offering patients protection and comfort.

**Delta Terry-Net® C** and **Delta Terry-Net® S** offer increased cushioning and protection for the patient. These products, along with **Delta Terry-Net® Thumb Spica** are consistently used in conjunction with Focused Rigidity Casting (FRC) techniques, where additional padding is not required.



# Jersey Standard

LATEX  
FREE  
FORMULA

## Natural Ribbed Stockinette


**Jersey standard** is a natural cotton stockinette. It is suitable for use with synthetic or plaster of Paris casting materials.

### Natural cotton

- Soft and gentle to the patients skin with reduced risk of skin irritation

### Autoclavable

- Compatible with steam and ethylene oxide sterilisation

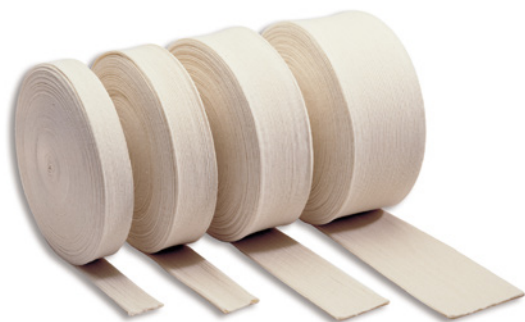

Code	Size	Colour	Pack Qty	NHS SC Code
72021-02	3cm x 25m roll	Natural	1 x 1	EGJ137
72021-03	5cm x 25m roll	Natural	1 x 1	EGJ138
72021-04	7cm x 25m roll	Natural	1 x 1	EGJ139
72021-05	10cm x 25m roll	Natural	1 x 1	EGJ140
72021-07	15cm x 25m roll	Natural	1 x 1	EGJ141
72021-08	20cm x 25m roll	Natural	1 x 1	EGJ142
72021-10	25cm x 25m roll	Natural	1 x 1	EGJ143

# Delta-Net® Stockinette

LATEX  
FREE  
FORMULA

## Synthetic Stockinette

**Delta-Net®** is a soft, comfortable, knitted, synthetic stockinette suitable for use with synthetic or plaster of Paris materials

### Synthetic material

- Stretches readily and easily conforms to body contours

### Soft, comfortable knit

- To protect the patient's skin under casting materials

### Easy to use dispenser

- Easily cut to length as required
- Easy storage and handling



Code	Size	Colour	Pack Qty	NHS SC Code
72307-05	2.5cm x 22.8m	White	1 x 1	EGJ207
72307-00	5.1cm x 22.8m	White	1 x 1	EGJ204
72307-01	7.6cm x 22.8m	White	1 x 1	EGJ205
72307-02	10.1cm x 22.8m	White	1 x 1	EGJ206



# Delta Terry-Net® C

LATEX  
FREE  
FORMULA

## Natural Terry Cloth Stockinette

**Delta Terry-Net® C** is a cotton towelling stockinette that can be used underneath either synthetic or plaster of Paris casting materials where extra cushioning and protection required for the patient.

Code	Size	Colour	Pack Qty
72305-00	5cm x 13.7m	Natural	1 x 1
72305-01	7.5cm x 13.7m	Natural	1 x 1
72305-02	10cm x 13.7m	Natural	1 x 1
72305-03	12.5cm x 13.7m	Natural	1 x 1
72305-04	15cm x 13.7m	Natural	1 x 1

### Natural cotton

- Soft and gentle to the patient's skin with reduced risk of skin irritation

### Versatile

- The cotton towelling cast liner is ideal when used in conjunction with Focused Rigidity Casting (FRC) techniques

### Easy to use dispenser

- Easily cut to length as required
- Easy storage and handling

### Autoclavable

- Compatible with steam and ethylene sterilisation



# Delta Terry-Net® S

LATEX  
FREE  
FORMULA

## Synthetic Terry Cloth Stockinette

**Delta Terry-Net® S** is a synthetic towelling stockinette that can be used underneath either synthetic or plaster of Paris casting materials where extra patient comfort is required.

Code	Size	Colour	Pack Qty
72306-00	5cm x 13.7m	White	1 x 1
72306-01	7.5cm x 13.7m	White	1 x 1
72306-02	10cm x 13.7m	White	1 x 1
72306-03	12.5cm x 13.7m	White	1 x 1

### Terry knit design

- Provides increased cushioning and protection for the patient

### Versatile

- Can be used in conjunction with Focused Rigidity Casting (FRC) techniques

### Easy to use dispenser

- Easily cut to length as required
- Easy storage and handling



# Delta Terry-Net® Thumb Spica

LATEX  
FREE  
FORMULA

## Thumb Spica Cast Synthetic Liner

**Delta Terry-Net®** cast liner is designed for use underneath scaphoid casts and other hand related injuries where extra patient comfort is required.

Code	Size	Colour	Pack Qty
72308-02	Short arm - 5cm	White	1 x 1
72308-03	Long arm - 5cm	White	1 x 1
72308-00	Short arm - 7cm	White	1 x 1
72308-01	Long arm - 7cm	White	1 x 1

### Terry knit design

- Provides increased cushioning and protection for the patient

### Integrated thumb piece

- Allows for quick and simple application

### Available in two arm lengths

- Fits a variety of patient sizes







## Thermoplastics


**Delta Thermo®** with its elastic memory, ensures the ability to return to its original sheet form and is ideally suited for serial splinting.



## Elastic Memory Thermoplastic

Delta Thermo® is ideally suited for serial splinting and splints for both the upper and lower limbs.

Code	Size	Pack Qty	NHS SC Code
72312-06	1.6mm thick non-perf, 60cm x 45cm	1 x 2 sheets	EAZ1105
72312-08	2.4mm thick non-perf, 60cm x 45cm	1 x 2 sheets	EAZ1107
72312-10	3.2mm thick non-perf, 60cm x 45cm	1 x 2 sheets	EAZ1109
72312-07	1.6mm thick perf, 60cm x 45cm	1 x 2 sheets	EAZ1106
72312-09	2.4mm thick perf, 60cm x 45cm	1 x 2 sheets	EAZ1108
72312-11	3.2mm thick perf, 60cm x 45cm	1 x 2 sheets	EAZ1110

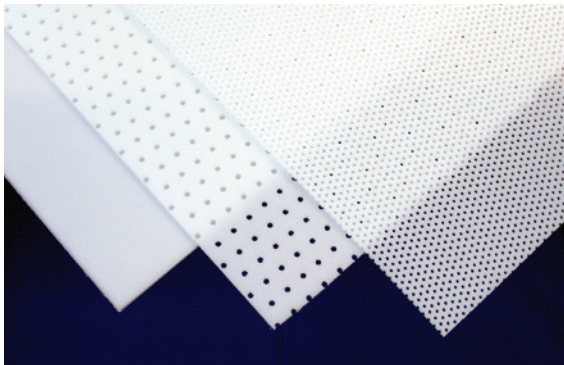

### Elastic memory

- Ensures ability to return to its original sheet form
- Easily remouldable
- Suitable for serial splinting

### Translucent appearance on reaching activation temperature

- Allows for precise positioning of the material on the area to be covered
- Prevents accidental self-bonding after heating

### Self-finishing edges

### Added patient comfort

### Activation

- Heating time 1 - 2 minutes at a temperature of 60 - 70°C/140 - 158°F
- Working time 3 - 4 minutes





## Casting Accessories

Essity offers a comprehensive range of accessories to complete the fracture management product range.

Focused Rigidity Casting (FRC) accessories such as **Delta Terry-Net® Adhesive Fleece Liner** complement the **Delta-Cast® Conformable** and **Delta Terry-Net®** stockinette products.

Adhesive and non-adhesive versions of **Tensoplast® Skin Traction Kits** are suitable for use with a variety of traction methods, and Essity also offer a range of cast shoes to provide essential protection for any cast.



## Pre-Tape Underwrap

**Tensoban®** is a very thin, polyurethane wrap, which can be used under adhesive bandages and tapes to protect the skin in taping and strapping techniques.

It can also be used as a pre-tape, applied on top of the padding layer in orthopaedic casting, to provide a smooth, compressed surface before the synthetic casting material is applied.

Code	Size	Colour	Pack Qty	NHS SC Code
71500-08	7cm x 20m roll	White	1 x 12	EAZ026



### Thin polyurethane pre-tape

- Results in a smooth, compressed surface on which to apply synthetic casting material

### Porous and water resistant

- Allows the skin to breathe
- Reduces the risk of skin maceration

### Versatile

- Particularly useful when creating 'windows' for inspection and for easy conversion of casts into resting splints

# Tensoplast® Sport

## Cast Edge Tape

**Tensoplast® Sport** elastic edge tape with tramline adhesive provides soft edging to casts and splints, and is particularly useful for edging removable Focused Rigidity Casts (FRC). The central, high-stretch section makes it easy to fold over cast edges.

Code	Size	Colour	Pack Qty	NHS SC Code
71548-00	3cm x 2.5m roll	White	1 x 1	EAZ027



### Tramline adhesive

- Cleanly and easily adheres to casting materials and thermoplastics

### Soft finish

- For increased patient comfort

### Central high stretch section

- Easy to manoeuvre the edging tape over the edge of the cast to produce a comfortable, neat finish to the cast



# Delta Terry-Net® Adhesive Felt Liner

LATEX  
FREE  
FORMULA

## Undercast Cushioning

**Delta Terry-Net® Adhesive Felt Liner** is a sheet of orthopaedic felt, used when additional patient comfort is required. The felt is ideal for providing protection to any bony prominences under a cast.

Code	Size	Colour	Pack Qty
72295-05	58cm x 100cm	White	1 x 1

### High quality adhesive backing


- Secures felt where you require it

### Soft finish

- For improved patient comfort

### Sheet format

- Can be cut to the desired size and shape as required

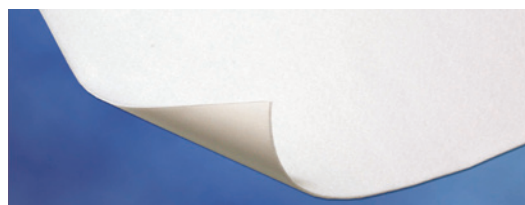

# Delta Terry-Net® Terry Cloth and Foam Liner

LATEX  
FREE  
FORMULA

## Undercast Cushioning

**Delta Terry-Net® Terry Cloth and Foam Liner** is a felt layer with foam sheet, which provides additional protection and cushioning for casting and bandaging techniques.

Code	Size	Colour	Pack Qty
72295-06	58cm x 100cm	White	1 x 1

### Soft finish

- For improved patient comfort

### Sheet format

- Can be cut to the desired size and shape as required





# Delta Terry-Net® Adhesive Fleece Liner

LATEX  
FREE  
FORMULA

## Cast Edging Fleece

Code	Size	Pack Qty	NHS SC Code
72995-07	3cm x 13.7m roll	1 x 1	EAZ071

**Delta Terry-Net® Adhesive Fleece Liner** provides a soft, padded edging to casts and splints and is particularly useful for edging removable Focused Rigidity Casts (FRC). With a high stretch it is easy to pad around corners and contours, giving a smooth, comfortable cast finish. It has a high quality adhesive for secure placement.



### High stretch

- Makes it easy to pad around corners and contours resulting in a neater, more comfortable cast for the patient

### High quality adhesive backing

- Secures fleece liner where you require it

### Soft finish

- For improved patient comfort

# Delta-Net® Adhesive Hook and Stretch Loop

LATEX  
FREE  
FORMULA

## Closure System

**Delta-Net® Adhesive Hook and Stretch Loop** is an easy to use product, particularly suited to Focused Rigidity Casting (FRC) techniques. The materials provide a strap closure system for removable casts and thermoplastics. The soft edges provide a comfortable fit for the patient.

Code	Size	Pack Qty	NHS SC Code
72295-00	Hook fabric 2.5cm x 9.1m	1 x 1	EAZ100
72295-02	Hook fabric 5cm x 9.1m	1 x 1	EAZ1384
72295-01	Stretch loop 2.5cm x 13.7m	1 x 1	EAZ101
72295-03	Stretch loop 5cm x 13.7m	1 x 1	EAZ1346



### High quality adhesive

- Readily adheres to casting materials and thermoplastics

### Soft edge stretch loop fabric

- Provides increased patient comfort

### Available in two widths

- Accommodates various casting requirements

### Easy to use dispenser

- Easily cut to length
- Easy storage and handling



## Vinyl Cast Shoes

**Solo®** is a versatile, hard-wearing cast shoe featuring an innovative rocker design and a longitudinally embedded steel strip for greater rigidity. The Solo® vinyl cast shoe has a textured upper surface and large heel cup to prevent movement. Soles and straps can be provided separately.

### Rocker sole design

- Allows a natural walking gait
- Limits hip problems during long- term immobilisation

### Cross-strap method

- Easy touch and close fastening
- Large heel cup to prevent movement on cast

### Colour coded straps according to size

- Easy recognition of required size

### Universal left / right

- Reduced stock holding required



Code	Size	Colour	Pack Qty	NHS SC Code
72298-40	X Small (With Straps)	-	1 x 10	CVL151
72298-41	Small (With Straps)	-	1 x 10	CVL152
72298-42	Medium (With Straps)	-	1 x 10	CVL153
72298-43	Large (With Straps)	-	1 x 10	CVL154
72298-44	X Large (With Straps)	-	1 x 10	CVL155
72298-12	X Small sole (20cm)	Blue	1 x 10	-
72298-13	Small sole (23cm)	Blue	1 x 10	CVL034
72298-14	Medium sole (25cm)	Blue	1 x 10	CVL038
72298-15	Large sole (28cm)	Blue	1 x 10	CVL059
72298-16	X Large sole (30cm)	Blue	1 x 10	-
72298-17	Short straps (40cm)	Blue loop	1 x 20	CVL035
72298-18	Medium straps (50cm)	Red loop	1 x 20	CVL036
72298-19	Long straps (60cm)	Green loop	1 x 20	CVL037

## Eco Canvas Cast Sandal

### Cast Shoes

The **Eco Canvas Cast Sandal** is constructed with only quality materials, using the highest manufacturing standards, yet at an economical price. The non-slip, tri-level, ribbed, rocker sole provides shock absorption and assists normal smooth walking gait. The three hook and loop closure straps ensure a secure fit.

### Tri-level ribbed rocker sole

- Provides shock absorption and assists smooth normal walking gait

### High level of durability

- Constructed with heavy canvas upper

### Secure fit

- Three hook and loop closure straps ensure a good fit

### Universal left and right

- Reduced stock holding required

Code	Size	Colour	Pack Qty	NHS SC Code
71427-23	XX small (16cm)	Blue canvas	1 x 1	-
71427-24	X small (19cm)	Blue canvas	1 x 1	CVL026
71427-25	Adult small (21cm)	Blue canvas	1 x 1	CVL025
71427-26	Adult medium (25cm)	Blue canvas	1 x 1	CVL024
71427-27	Adult large (29cm)	Blue canvas	1 x 1	CVL023
71427-28	Adult X large (31cm)	Blue canvas	1 x 1	CVL098





## Skin Traction Kits

**Tensoplast® STK** is a ready to use kit, including all items needed for the secure application of sustained traction weight support, and suitable for use with a variety of traction methods.

Tensoplast® STK is available in an adhesive and non-adhesive version.

Code	Size	Pack Qty	NHS SC Code
71544-00	Adhesive: adult	1 x 12	ESB151
71554-00	Adhesive: child	1 x 12	ESB153
71555-00	Non-adhesive: adult	1 x 12	ESC051
71556-00	Non-adhesive: child	1 x 12	ESC805



### Adhesive version

- Clear, low-allergy, pressure-sensitive adhesive for secure, reliable adhesion to the limb

### Non-adhesive version

- Foam skin contact layer, breathable and suitable for tender skin

### Padded foot plates

- Comfortable support of the foot

### Conformable elastic bandage

- Ease of application to give secure positioning





Delta-Cast<sup>®</sup>,  
an Essity brand



**Essity**  
T/A BSN medical Limited  
PO Box 258 • Willerby • Hull • HU10 6WT

[www.medical.essity.co.uk](http://www.medical.essity.co.uk)  
Tel: 01482 670100 • Fax: 01482 670111  
E-mail: [orders.uk@essity.com](mailto:orders.uk@essity.com)