

# JOBST® FarrowWrap® Measuring step-by-step guide

To measure for a JOBST FarrowWrap, you will need



A pen



An order form



A measuring tape

Once you have the measurements, the order form will guide you on which size to order

Ready-to-wear order form and prescription request

For order to Customer Services call 0845 122 3450  
Email order to Customer Services at: [comprex@uk.jobst.com](mailto:comprex@uk.jobst.com)

Date: \_\_\_\_\_ Purchase Order No.: \_\_\_\_\_ Patient Number: \_\_\_\_\_

Measured By: \_\_\_\_\_ Tel: \_\_\_\_\_ Email: \_\_\_\_\_

Delivery Address: \_\_\_\_\_ Invoice Address: \_\_\_\_\_

### Measuring instructions

**Thighpiece - 3 simple measuring points**

- Measure point A - the widest part of the thigh (circumference at the widest part).
- Use chart below to determine size at the top of the column of 3 measurements.
- Measure the length of the thigh (circumference at the widest part).

**Legpiece - 2 simple measuring points**

- Measure the widest part of the calf (circumference at the widest part).
- Use chart below to determine size at the top of the column of 2 measurements.

**Footpiece - 2 simple measuring points**

- Measure A (circumference at the widest part of the foot).
- Use chart below to determine size at the top of the column of 2 measurements.

	Small	Small	Medium	Large	Large
Thighpiece (circumference)	40-45cm	45-50cm	50-55cm	55-60cm	60-65cm
Legpiece (circumference)	35-40cm	40-45cm	45-50cm	50-55cm	55-60cm
Footpiece (circumference)	20-25cm	25-30cm	30-35cm	35-40cm	40-45cm

**Thighpiece**

Size: ☐ Small ☐ Medium ☐ Large

Length A-D: ☐ Short ☐ Regular ☐ Long

HT code: ☐ 1 ☐ 2 ☐ 3 ☐ 4 ☐ 5 ☐ 6 ☐ 7 ☐ 8 ☐ 9 ☐ 10

Code: ☐ 1 ☐ 2 ☐ 3 ☐ 4 ☐ 5 ☐ 6 ☐ 7 ☐ 8 ☐ 9 ☐ 10

Quantity Required:

Repeat prescription request every:  weeks

**Legpiece**

Size: ☐ Small ☐ Medium ☐ Large

Length A-D: ☐ Short ☐ Regular ☐ Long

HT code: ☐ 1 ☐ 2 ☐ 3 ☐ 4 ☐ 5 ☐ 6 ☐ 7 ☐ 8 ☐ 9 ☐ 10

Code: ☐ 1 ☐ 2 ☐ 3 ☐ 4 ☐ 5 ☐ 6 ☐ 7 ☐ 8 ☐ 9 ☐ 10

Quantity Required:

Repeat prescription request every:  weeks

**Footpiece**

Size: ☐ Small ☐ Medium ☐ Large

Length A-D: ☐ Short ☐ Regular ☐ Long

HT code: ☐ 1 ☐ 2 ☐ 3 ☐ 4 ☐ 5 ☐ 6 ☐ 7 ☐ 8 ☐ 9 ☐ 10

Code: ☐ 1 ☐ 2 ☐ 3 ☐ 4 ☐ 5 ☐ 6 ☐ 7 ☐ 8 ☐ 9 ☐ 10

Quantity Required:

Repeat prescription request every:  weeks

**JOBST® FarrowWrap®**

Size: ☐ Small ☐ Medium ☐ Large

Length: ☐ Short ☐ Regular ☐ Long

HT code: ☐ 1 ☐ 2 ☐ 3 ☐ 4 ☐ 5 ☐ 6 ☐ 7 ☐ 8 ☐ 9 ☐ 10

Code: ☐ 1 ☐ 2 ☐ 3 ☐ 4 ☐ 5 ☐ 6 ☐ 7 ☐ 8 ☐ 9 ☐ 10

Quantity Required:

Repeat prescription request every:  weeks

8 pair of open-toe slippers will be provided free of charge with each JOBST® FarrowWrap® order and stock.

Below shoe codes with other items, high-top codes with 10 slippers.

Customer Services 0845 122 3450 Compression Therapy England 0800 369 8438  
Email: [comprex@uk.jobst.com](mailto:comprex@uk.jobst.com) Website: [www.jobst.co.uk](http://www.jobst.co.uk)

# Measuring for the JOBST FarrowWrap

FOOTPIECE

## INTRODUCTION


Two measurements required



Length



Circumference

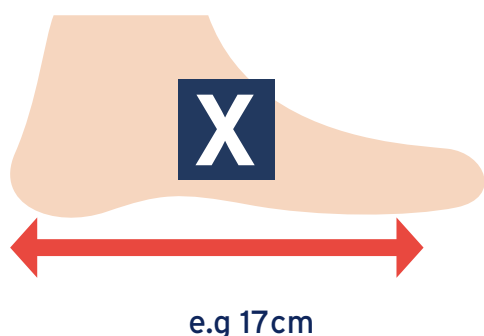


## FOOT LENGTH

Straight measurement from back of the heel to base of the **little toe**

This is measurement **X**

## MID-FOOT CIRCUMFERENCE

The midfoot measurement is a circumferential measurement taken at the midfoot point

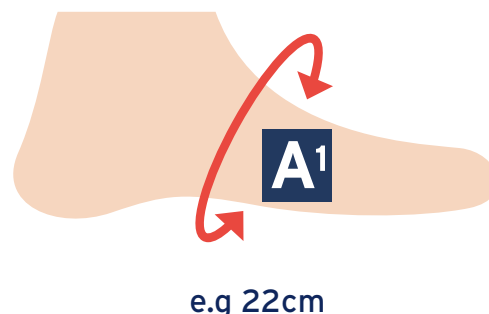

This is measurement **A1**

## HOW TO USE THE FORM

- Mark all sizes your measurement fits into
- Choose the column with the most values in (X Small to X Large)
- Choose which length (X) row has the most values in

In this example it's an **X-SMALL**  
**REGULAR** length fit


	X Small	Small	Medium	Large	X Large
<b>A1</b> Mid-foot	22-24cm	25-27cm	28-30cm	31-34cm	35-40cm
<b>X</b> Regular	15-17cm	16-19cm	18-20cm	19-22cm	21-24cm
<b>X</b> Tall	18-20cm	20-22cm	21-23cm	23-25cm	25-27cm

# Measuring for the JOBST FarrowWrap

## LEGPiece

### INTRODUCTION

Three measurements  
required

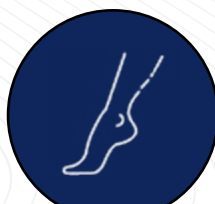


Calf  
circumference



Ankle  
circumference



Thigh  
length

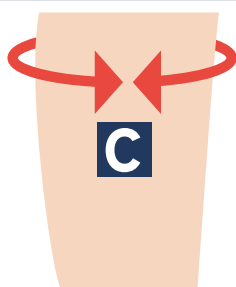

e.g 37cm

### CALF CIRCUMFERENCE

Taken at the widest part of the calf

This is measurement C

### ANKLE CIRCUMFERENCE

Taken at the narrowest part of the ankle.

This is measurement B



e.g 22cm

B



e.g 38cm

### LEG LENGTH

Measure from the back of the leg, from the  
floor to 2cm below the crease of the knee,  
following the contour of the leg

This is measurement A-D

### HOW TO USE THE FORM

- Mark all sizes your measurement fits into
- Choose the column with the most values in (X Small to X Large)
- Choose which length (A-D) row has the most values in

	X Small	Small	Medium	Large	X Large
<b>C</b> Mid-calf	36-43cm	42-50cm	48-58cm	53-63cm	58-68cm
<b>B</b> Ankle	21-25cm	25-30cm	30-36cm	36-42cm	42-50cm
<b>A-D</b> Regular	35-39cm	37-41cm	39-43cm	41-45cm	42-46cm
<b>A-D</b> Tall	40-44cm	42-46cm	44-48cm	46-50cm	47-51cm

In this example it's an X-SMALL REGULAR length fit

# Measuring for the JOBST FarrowWrap

## THIGHPIECE

### INTRODUCTION

Three measurements  
required




Above knee  
circumference



Groin  
circumference



Thigh  
length

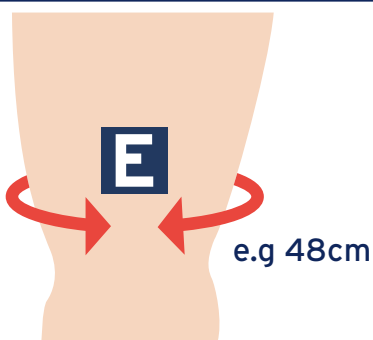

### ABOVE KNEE CIRCUMFERENCE



Measure 5cm above the knee

This is measurement E

### GROIN CIRCUMFERENCE

Measure at the gluteal fold (where the  
buttock meets the leg)

This is measurement G

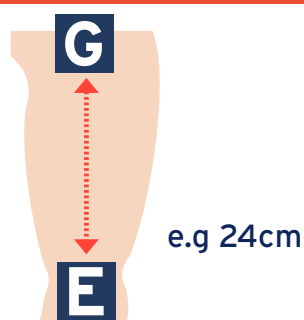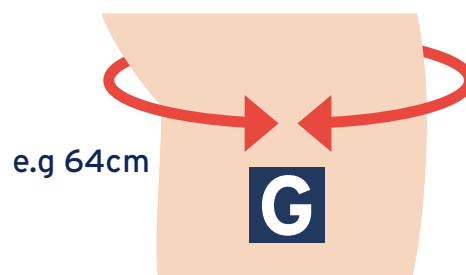

### THIGH LENGTH

Measure the length of the thigh medially  
(inside leg) between E and G following  
the contour of the leg

This is measurement E-G

### HOW TO USE THE FORM

- Mark all sizes your measurement fits into
- Choose the column with the most values in (X Small to X Large)
- Choose which length (E-G) row has the most values in

	X-Small	Small	Medium	Large	X Large
<b>G Groin</b>	60-70cm	65-75cm	70-80cm	75-90cm	85-100cm
<b>E Above knee</b>	47-52cm	53-59cm	60-66cm	67-73cm	74-80cm
<b>E-G Short</b>	16-20cm	16-20cm	16-20cm	16-20cm	16-20cm
<b>E-G Regular</b>	21-26cm	21-26cm	21-26cm	21-26cm	21-26cm
<b>E-G Tall</b>	27-31cm	27-31cm	27-31cm	27-31cm	27-31cm

In this example it's an X-SMALL REGULAR length fit



# FINALISING YOUR ORDER

You should now have all your measurements and have been able to translate these into the sizes required

**JOBST® FarrowWrap® Order Codes**

Code	Size	Length	Width	Foot
JOBST® FarrowWrap® L (20-30cm)				
JOBST® FarrowWrap® L (30-40cm)				
JOBST® FarrowWrap® L (40-50cm)				
JOBST® FarrowWrap® L (50-60cm)				
JOBST® FarrowWrap® L (60-70cm)				
JOBST® FarrowWrap® L (70-80cm)				
JOBST® FarrowWrap® L (80-90cm)				
JOBST® FarrowWrap® L (90-100cm)				
JOBST® FarrowWrap® L (100-110cm)				
JOBST® FarrowWrap® L (110-120cm)				
JOBST® FarrowWrap® L (120-130cm)				
JOBST® FarrowWrap® L (130-140cm)				
JOBST® FarrowWrap® L (140-150cm)				
JOBST® FarrowWrap® L (150-160cm)				
JOBST® FarrowWrap® L (160-170cm)				
JOBST® FarrowWrap® L (170-180cm)				
JOBST® FarrowWrap® L (180-190cm)				
JOBST® FarrowWrap® L (190-200cm)				
JOBST® FarrowWrap® L (200-210cm)				
JOBST® FarrowWrap® L (210-220cm)				
JOBST® FarrowWrap® L (220-230cm)				
JOBST® FarrowWrap® L (230-240cm)				
JOBST® FarrowWrap® L (240-250cm)				
JOBST® FarrowWrap® L (250-260cm)				
JOBST® FarrowWrap® L (260-270cm)				
JOBST® FarrowWrap® L (270-280cm)				
JOBST® FarrowWrap® L (280-290cm)				
JOBST® FarrowWrap® L (290-300cm)				
JOBST® FarrowWrap® L (300-310cm)				
JOBST® FarrowWrap® L (310-320cm)				
JOBST® FarrowWrap® L (320-330cm)				
JOBST® FarrowWrap® L (330-340cm)				
JOBST® FarrowWrap® L (340-350cm)				
JOBST® FarrowWrap® L (350-360cm)				
JOBST® FarrowWrap® L (360-370cm)				
JOBST® FarrowWrap® L (370-380cm)				
JOBST® FarrowWrap® L (380-390cm)				
JOBST® FarrowWrap® L (390-400cm)				
JOBST® FarrowWrap® L (400-410cm)				
JOBST® FarrowWrap® L (410-420cm)				
JOBST® FarrowWrap® L (420-430cm)				
JOBST® FarrowWrap® L (430-440cm)				
JOBST® FarrowWrap® L (440-450cm)				
JOBST® FarrowWrap® L (450-460cm)				
JOBST® FarrowWrap® L (460-470cm)				
JOBST® FarrowWrap® L (470-480cm)				
JOBST® FarrowWrap® L (480-490cm)				
JOBST® FarrowWrap® L (490-500cm)				
JOBST® FarrowWrap® L (500-510cm)				
JOBST® FarrowWrap® L (510-520cm)				
JOBST® FarrowWrap® L (520-530cm)				
JOBST® FarrowWrap® L (530-540cm)				
JOBST® FarrowWrap® L (540-550cm)				
JOBST® FarrowWrap® L (550-560cm)				
JOBST® FarrowWrap® L (560-570cm)				
JOBST® FarrowWrap® L (570-580cm)				
JOBST® FarrowWrap® L (580-590cm)				
JOBST® FarrowWrap® L (590-600cm)				
JOBST® FarrowWrap® L (600-610cm)				
JOBST® FarrowWrap® L (610-620cm)				
JOBST® FarrowWrap® L (620-630cm)				
JOBST® FarrowWrap® L (630-640cm)				
JOBST® FarrowWrap® L (640-650cm)				
JOBST® FarrowWrap® L (650-660cm)				
JOBST® FarrowWrap® L (660-670cm)				
JOBST® FarrowWrap® L (670-680cm)				
JOBST® FarrowWrap® L (680-690cm)				
JOBST® FarrowWrap® L (690-700cm)				
JOBST® FarrowWrap® L (700-710cm)				
JOBST® FarrowWrap® L (710-720cm)				
JOBST® FarrowWrap® L (720-730cm)				
JOBST® FarrowWrap® L (730-740cm)				
JOBST® FarrowWrap® L (740-750cm)				
JOBST® FarrowWrap® L (750-760cm)				
JOBST® FarrowWrap® L (760-770cm)				
JOBST® FarrowWrap® L (770-780cm)				
JOBST® FarrowWrap® L (780-790cm)				
JOBST® FarrowWrap® L (790-800cm)				
JOBST® FarrowWrap® L (800-810cm)				
JOBST® FarrowWrap® L (810-820cm)				
JOBST® FarrowWrap® L (820-830cm)				
JOBST® FarrowWrap® L (830-840cm)				
JOBST® FarrowWrap® L (840-850cm)				
JOBST® FarrowWrap® L (850-860cm)				
JOBST® FarrowWrap® L (860-870cm)				
JOBST® FarrowWrap® L (870-880cm)				
JOBST® FarrowWrap® L (880-890cm)				
JOBST® FarrowWrap® L (890-900cm)				
JOBST® FarrowWrap® L (900-910cm)				
JOBST® FarrowWrap® L (910-920cm)				
JOBST® FarrowWrap® L (920-930cm)				
JOBST® FarrowWrap® L (930-940cm)				
JOBST® FarrowWrap® L (940-950cm)				
JOBST® FarrowWrap® L (950-960cm)				
JOBST® FarrowWrap® L (960-970cm)				
JOBST® FarrowWrap® L (970-980cm)				
JOBST® FarrowWrap® L (980-990cm)				
JOBST® FarrowWrap® L (990-1000cm)				

All product and PIP codes for the JOBST FarrowWrap variants can be found on the reverse of the order form

The description for the garment can then be used to raise a prescription. E.G. JOBST FarrowWrap Strong; footpiece - xsmall, regular; legpiece - xsmall, regular; thighpiece - xsmall, regular

## SUPPORT CONTACT INFORMATION

Customer services contact number: **0845 1223450**

Customer services email: **compression.uk@jobst.com**

Concierge service email: **concierge.service@essity.com**

Website: **bsnmedical.co.uk**

Twitter: **@JOBSTUK**